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## EC-Declaration of Conformity

**Manufacturer:** CSM GRAF GmbH  
Aachweg 7, CH-9323 Steinach

**Authorised representative:** Medical Technology  
Promedt Consulting GmbH, Altenhofstrasse 80,  
D-66386 St. Ingbert

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This declaration of conformity is issued under the sole responsibility of CSM Graf GmbH. We hereby declare that the medical device(s) specified above meet the provision of the Regulation (EU) MDR 2017/745 for medical devices.

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
ISO 10993-1; ISO 10993-3; ISO 13485

The following **harmonised standards**, amongst others, were brought into use in order to **evaluate the products** with regard to their conformity with the requirements contained in the specified statutory regulations:

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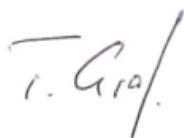This declaration is submitted on behalf of the above-mentioned manufacturer by:

Thomas Graf, Managing Director  
Steinach, 16.09.2021

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## Szalay Cyto-Spatula instruction for use

### General information

The Szalay Cyto-Spatula is a collection instrument for the traditional Pap smear from the cervix uteri for cancer screening. Four different shapes of the Szalay Cyto-Spatula are available depending on the shape and size of the portio. See [www.csmgraf.ch](http://www.csmgraf.ch)

### Important general features of the Szalay Cyto-Spatula

- The Szalay Cyto-Spatula is a **medical device** intended for professional use only. Usually by the gynecologist or a physician.
- **Non** sterile medical device
- Intended for **single use** only and to be disposed of immediately after use

### Instructions for using the SZALAY CYTO-SPATULA

1. Have the SZALAY CYTO-SPATULA, microscope slide and fixative ready near the examination couch.
2. Inscribe the patient's name or number onto the slide.
3. Position a speculum to expose the uterine cervix so that the smear can be taken under direct vision (with a colposcope whenever possible!).
4. Gently remove any mucus and debris from the surface of the cervix, using a cotton swab.
5. Insert the pointed tongue of the spatula into the cervical canal until the broad shoulder of the spatula lies completely against the cervix. Begin at the "3 o'clock" position and, exerting gentle pressure, rotate the spatula through a complete 360 degree turn in a clockwise direction.

#### Choice of SZALAY-CYTO-SPATULA:

Spatula No. 1: for a small or medium-sized cervix (most cases).

Spatula: No. 2 for a larger cervix with a wide cervical os.

Spatula No. 3 as for No. 2

Spatula No. 4: a) for a larger cervix with an extended transformation zone, in addition to a smear with a No. 1, 2 or 3 spatula. b) for a smear from the outer surface only, in patients with an obliterated cervical canal or one that is too narrow for the passage of a No. 1 spatula. The cytology laboratory must be informed if this is the case.

6. Holding the SZALAY-CYTO-SPATULA with the shoulder in front, spread the material in several light strokes, always parallel, along the entire length of the slide.
7. Fix the thinly- and evenly-smearred material on the slide immediately, whilst it is still moist.
8. After fixation, the slide is placed in the shipping case with the smear side up.
9. Discard the Szalay spatula after use.
10. Video instructions under: [www.csmgraf.ch](http://www.csmgraf.ch)

### Important notice

Slight bleeding may occur during extraction of the material in 2-3% of cases, which however will not interfere with the subsequent colposcopic examination. If the patient is duly forewarned, slight spotting for 1-2 days is not unduly worrisome. Only very rarely does the blood affect the cytologic assessment. The colposcopic examination may also be conducted prior to taking the cervical smear! If the smear is taken around the middle of the menstrual cycle, bleeding is extremely rare. Otherwise, bleeding can occur with all smear instruments.

Any problems with the use of Szalay Cyto-Spatula should be reported immediately to the manufacturer and the authorities in your country.

## Hersteller / Manufacturer

CSM Graf GmbH  
Aachweg 7  
CH-9323 Steinach  
[www.csmgraf.ch](http://www.csmgraf.ch)